



**Vaporized Hydrogen Peroxide/Plasma (VH<sub>2</sub>O<sub>2</sub>) Indicator Ink for Flexographic Printing Processes**

**REF** IPBP-01

True Indicating's VH<sub>2</sub>O<sub>2</sub> Ink, Product Code IPBP-01, is a flexographic ink formulation used to monitor exposure VH<sub>2</sub>O<sub>2</sub> conditions. The ink is a water based formula which contains no heavy metals or other harmful materials and has low VOC's.

<b>Physical Properties</b>	
Process	Vaporized Hydrogen Peroxide VH <sub>2</sub> O <sub>2</sub>
Ink Carrier	Water
Packaging	HDPE or equivalent containers
pH	8.5 – 10.0
Viscosity Zahn Cup # 2	15 - 30 seconds
Ink Appearance	Thick pink liquid
Printed Indicator Color	Initial Color: Blue Signal Color: Pink
Anilox Roller	200 – 100 lpi
Flexographic Line Speed	≥150 ft./min.

<b>Intended Use</b>	
Indicator Ink is intended for manufacture VH <sub>2</sub> O <sub>2</sub> Process and Multi-Critical Variable Indicators. The Indicators are intended for use in distinguishing processed from unprocessed goods or to indicate exposure to a sterilization process at a stated value (SV). Validation of Indicators and their performance is at the discretion of the manufacturer. The Indicator Ink has been confirmed to be ISO 11140-1:2014 Type 1 compliant on Tyvek® or Film when exposed per the conditions outlined below.	
Type 1 Process Indicator	2.3 mg/L VH <sub>2</sub> O <sub>2</sub> at 50°C for 6 minutes
Printed Indicators manufactured with ink IPBP-01 can be used at temperatures between 30° and 80°C.	



True Indicating, LLC  
 946 Kane Street, Suite A  
 Toledo OH 43612-1372  
 T: 419-476-7119  
[www.TrueIndicating.com](http://www.TrueIndicating.com)

Email: [info@trueindicating.com](mailto:info@trueindicating.com)





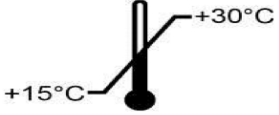


# Technical Data Sheet

## Instructions for Use

IPBP-01 is intended for flexographic printing onto Tyvek® or film based substrates.  $\text{VH}_2\text{O}_2$  exposure of printed ink will transition from Blue to Pink. The shade of initial and signal color will depend on the thickness of ink printed, the substrate onto which it is printed, and the exposure conditions.

Colors shown are examples of actual printed ink using a 165 lpi Anilox roller and Tyvek®. Signal or end point color will vary due to printing conditions and exposure variables.

Initial Color	Signal Color
	

Storage and Shelf Life	
<b>Shelf Life</b>	4 months from date of manufacture
	15°C to 30°C (Protect from heat sources)
	Keep away from sterilant and other chemical agents. <b>Do not reuse ink!</b> Discard all remaining ink. Up to 5% by weight of water or ether can be added to the ink to decrease dry time.
	20% to 80% Relative Humidity

## Disposal

Dispose in accordance with all federal, state, provincial, and local regulations. This material does not meet the criteria of a hazardous waste as defined under the Resource Conservation and Recovery Act (RCRA) 40 CFR 261.



True Indicating, LLC  
 946 Kane Street, Suite A  
 Toledo OH 43612-1372  
 T: 419-476-7119  
[www.TrueIndicating.com](http://www.TrueIndicating.com)

Email: [info@trueindicating.com](mailto:info@trueindicating.com)